

Myco 5-in-1 PLUS with Vertu TOUCH: Detect **SIX** mycotoxins with **ONE** extraction.

1

Weigh 5 g ± 0.1 g finely-ground sample into Vertu PREP tube. Add 25 mL AQUA Premix to extraction tube.



2

Set Vertu PREP mixer for 30 seconds. Close the tube and twist to lock it on the mixer. The mixer will start automatically



3

Filter the extract into a V Filter Cup for no more than 5 minutes.



Afla-V™
QUANTITATIVE STRIP TESTS

Ochra-V™
QUANTITATIVE STRIP TESTS


Fumo-V™
QUANTITATIVE STRIP TESTS

Zearala-V™
QUANTITATIVE STRIP TESTS

T2/HT2-V™
QUANTITATIVE STRIP TESTS

DON-V™
QUANTITATIVE STRIP TESTS

Pipette 100 µL of extract to the sample well on the strip at 1 drop/second.

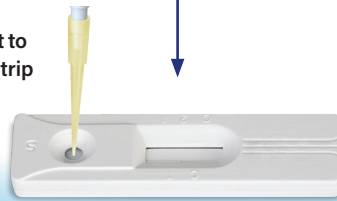

Place DON-V™ strip into the mini-incubator at 40 °C and close the lid. Incubate strip for 2–10 minutes. This can be started while doing the sample extraction and filtration.



Pipette 100 µL of extract to the sample well on the DON-V strip at 1 drop/second and wait for the strip to develop for 5 minutes, leaving it inside the incubator (40 °C).



Solvent-free extraction!

Eliminate the expense and environmental hazards of toxic organic solvents.

Place the strip on a flat surface for 5 minutes.

While waiting for 5 min, prepare Vertu TOUCH calibration by scanning the corresponding Vertu TOUCH barcode for the specific mycotoxin kit and lot#*



Place the strip inside the Vertu TOUCH (the round opening in first), then press [Start Measurement].



- ▶ Press [Print] to print the result.
- ▶ Press [Same Barcode] to run the next test using the same lot of strips.
- ▶ Press [Change Barcode] to change the method or lot of strips being used.

* Calibration only needs to be scanned once per lot. Instrument stores calibration.

For more information visit www.vicam.com

VICAM, A Waters Business Headquarters:

34 Maple Street Milford, MA 01757, USA ■ Tel.: +1 800 338 4381 ■ +1 508 482 4935 ■ Fax.: +1 508 482 4972